



## Sternal Closure System Implant Positioner (P/N 201.047.47)

Note: This cleaning instruction is intended to augment JACE Medical CS.0001 Product Care, Cleaning, and Sterilization instructions and is not intended as a replacement for the validated parameters contained therein.

### 1 Depress Buttons

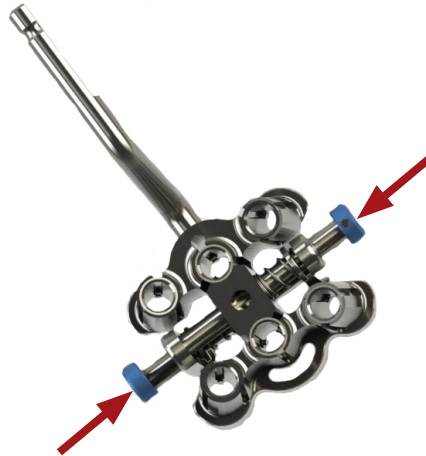

### 2 Flush Springs with Buttons Depressed

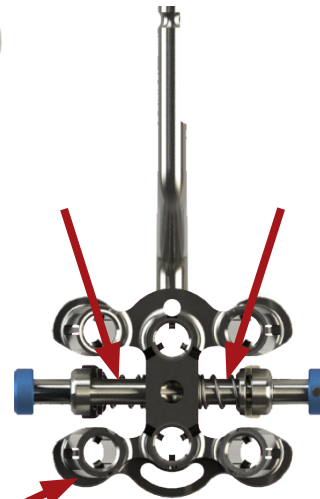

### 3 Flush Bores (Repeat for all 6 Bores)



Detail View of Bores

### 4 Clean and Sterilize per JACE Medical CS.0001 Product Care, Cleaning, and Sterilization instructions.



JACE Medical  
3516 Commerce Drive  
Warsaw, IN 46580  
USA  
Ph 574-306-0355



**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands



CS.5001  
Rev C