



Sternal Closure System MIS Implant Positioner (P/N 201.057.47)

Note: This cleaning instruction is intended to augment JACE Medical CS.0001 Product Care, Cleaning, and Sterilization instructions and is not intended as a replacement for the validated parameters contained therein.

1 Depress Buttons

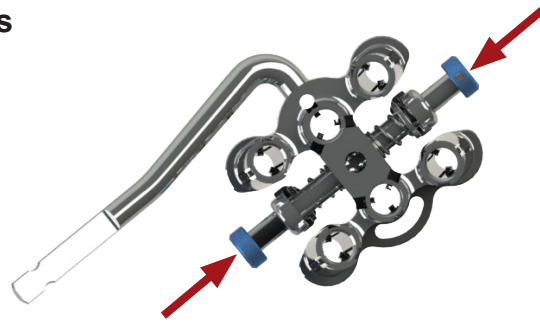

2 Flush Springs with Buttons Depressed

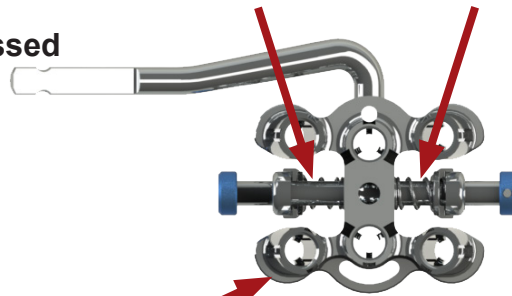

3 Flush Bores (Repeat for all 6 Bores)



Detail View of Bores

4 Clean and Sterilize per JACE Medical CS.0001 Product Care, Cleaning, and Sterilization instructions.



MANUFACTURER:

JACE Medical
3516 Commerce Drive
Warsaw, IN 46580
USA
Ph 574-306-0355

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

CE
0088

CS.5004
Rev B



JACE
MEDICAL