


## Grand Pre Plate Screw Depth Guide (P/N 201.035.00)

Note: This cleaning instruction is intended to augment JACE Medical CS.0001 Product Care, Cleaning, and Sterilization instructions and is not intended as a replacement for the validated parameters contained therein.


**1 Remove Item 1 from Item 2 by hand**



**2 Flush slots**

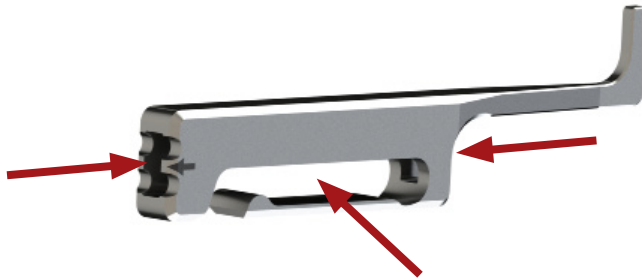

**3 Insert item 1 into item 2 by hand**

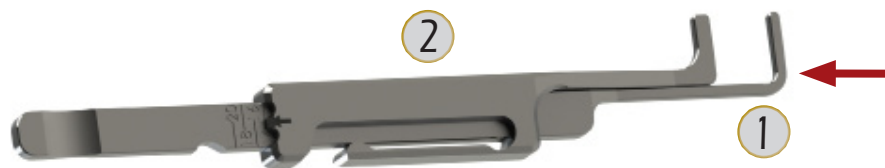

**4 Clean and Sterilize per JACE Medical CS.0001 Product Care, Cleaning, and Sterilization instructions.**



**MANUFACTURER:**

JACE Medical  
3516 Commerce Drive  
Warsaw, IN 46580  
USA  
Ph 574-306-0355



**Emergo Europe**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands



CS.5005  
Rev B